


# SaliVIR Collect Kit



Made in the UK. Chronomics Limited, 1 St James Court, Norwich, NR3 1RU, UK.

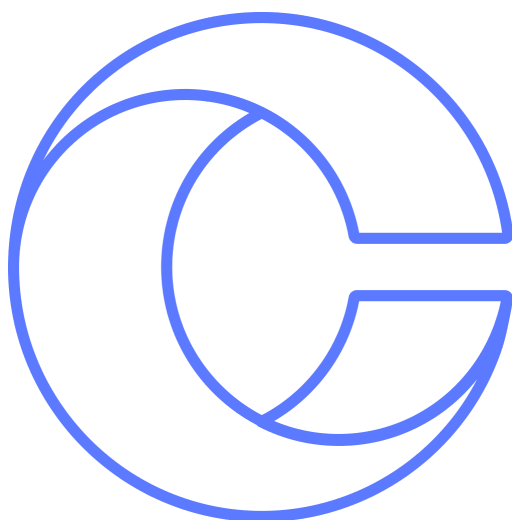

C1COV00400



For in Vitro Diagnostic Use.



Wash with water if stabilizing liquid comes in contact with eyes or skin. Do not ingest. Small cap may pose a choking hazard.



Store at 15-30°.






Consult package insert.



## Chronomics sample collection guide v1.0.1

### SaliVIR Collect Kit

#### This kit includes:

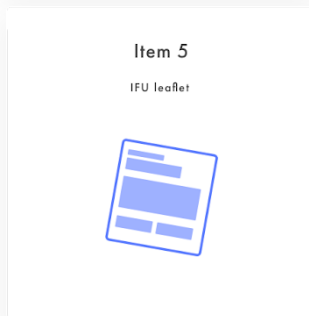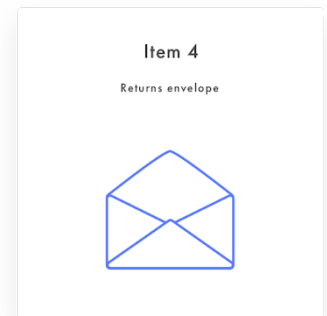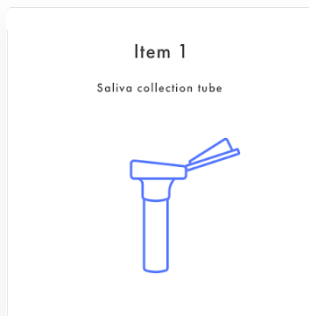











#### Intended use:

To collect a saliva sample for the purpose of *in vitro* diagnostic testing in a certified laboratory. This kit can be used for saliva self-collection with or without the supervision of a healthcare professional.

#### Precautions:

- You will need internet access to register the barcode on your saliva collection tube.
- Do not eat, drink, chew gum, brush your teeth or smoke for at least 30 minutes before providing your sample.
- Do not remove plastic film from the funnel lid.

Self-collection kit instructions:

<p><b>Step 1</b> Wash your hands</p> <p>Wash and dry hands before opening the kit.</p> 	<p><b>Step 2</b> Register your kit</p> <p>When you are ready to take your sample, register your kit online and enter the 14-digit barcode found on your saliva collection tube.</p> <p><a href="https://app.chronomics.com/register-covid">app.chronomics.com/register-covid</a></p>	<p><b>Step 3</b> Open the tube and fill</p> <p>Remove the tube packaging and spit into the tube container until the amount of saliva reaches the fill line. Foam and bubbles don't count as saliva.</p> 	<p><b>Step 4</b> Close the tube</p> <p>Close the funnel cap until you hear a click to release the stabilising fluid.</p> 
<p><b>Step 5</b> Remove funnel cap</p> <p>Unscrew the funnel cap and screw on the smaller blue cap provided.</p> 	<p><b>Step 6</b> Shake</p> <p>Shake the tube vigorously for at least 30 seconds to mix your saliva sample with the stabilising fluid.</p> 	<p><b>Step 7</b> Wash your hands again</p> <p>Wash and dry hands again before packaging your sample.</p> 	<p><b>Step 8</b> Place tube in the bag</p> <p>Wrap the tube in absorbent material (paper towel, cotton wool or tissue), insert into the specimen transportation bag and seal it.</p> 
<p><b>Step 9</b> Place bag in the box</p> <p>Insert the specimen transportation bag back into the cardboard box. Multiple samples can be inserted into one box as long as they are individually sealed in specimen bags.</p> 	<p><b>Step 10</b> Place box in the envelope</p> <p>Insert the cardboard box into the pre-paid returns envelope.</p> 	<p><b>Step 11</b> Book your collection</p> <p>Follow the instructions provided via email to book your collection.</p> 	<p><b>Step 12</b> Finally!</p> <p>If your collection is not collected on the same day, freeze or refrigerate your sample to prevent degradation.</p> 