



SaliVIR Collect kit

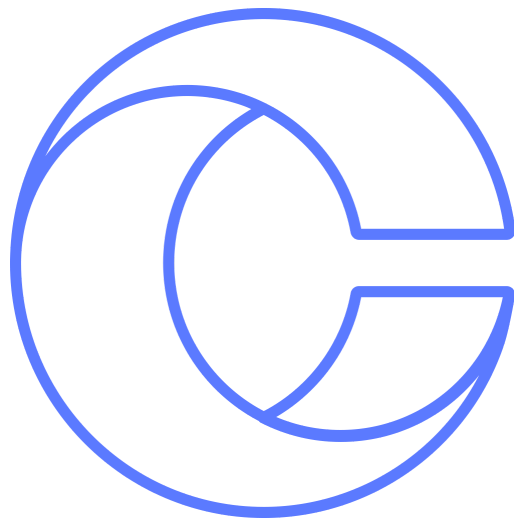

Chronomics sample collection guide

SaliVIR Collect Kit

This kit includes:

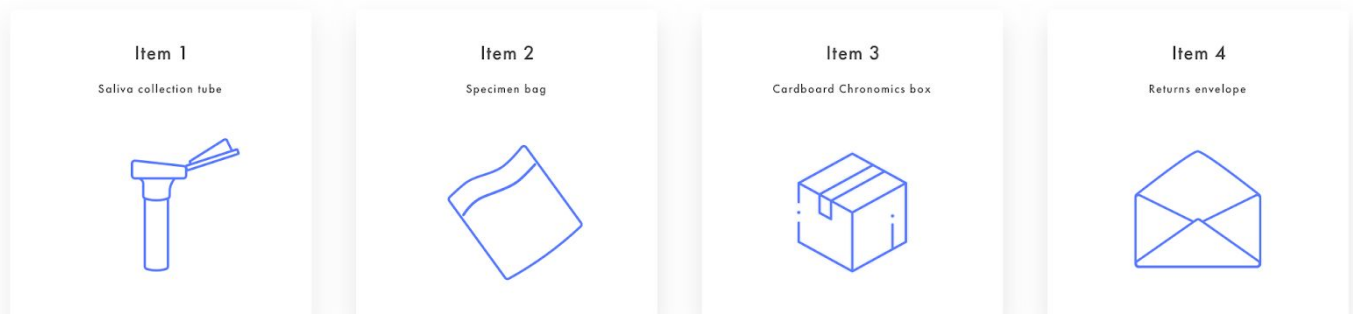

Intended use:

To collect a saliva sample for the purpose of *in vitro* diagnostic testing in a certified laboratory. This kit can be used for saliva self-collection with or without the supervision of a healthcare professional.

Chronomics sample collection guide

SaliVIR Collect Kit

Self-collection kit instructions:

Step 1

Wash your hands

Wash and dry hands before opening the kit.



Step 2

Register your kit

When you are ready to take your sample, register your kit online and enter the 14-digit barcode found on your saliva collection tube.

app.chronomics.com/register-covid

Step 3

Open the tube and fill

Remove the tube packaging and spit into the tube container until the amount of saliva reaches the fill line. Foam and bubbles don't count as saliva.



Step 4

Close the tube

Close the funnel cap until you hear a click to release the stabilising fluid.



Step 5

Remove funnel cap

Unscrew the funnel cap and screw on the smaller blue cap provided.



Step 6

Shake

Shake the tube vigorously for at least 30 seconds to mix your saliva sample with the stabilising fluid.



Step 7

Wash your hands again

Wash and dry hands again before packaging your sample.



Step 8

Place tube in the bag

Wrap the tube in absorbent material (paper towel, cotton wool or tissue), insert into the specimen transportation bag and seal it.



Step 9

Place bag in the box

Insert the specimen transportation bag back into the cardboard box. Multiple samples can be inserted into one box as long as they are individually sealed in specimen bags.



Step 10

Place box in the envelope

Insert the cardboard box into the pre-paid returns envelope.



Step 11

Book your collection

Follow the instructions provided via email to book your collection.



Step 12

Finally!

If your collection is not collected on the same day, freeze or refrigerate your sample to prevent degradation.



